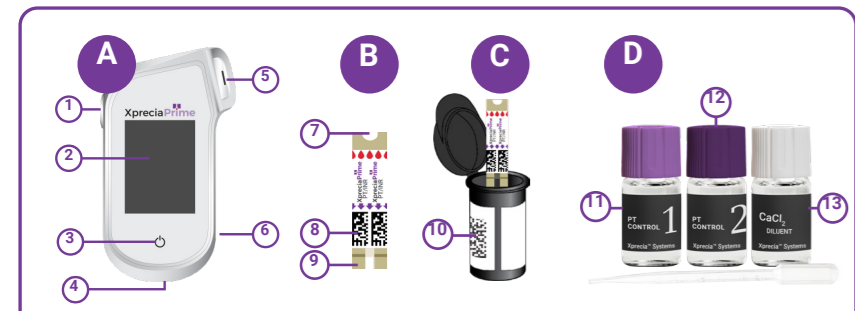
Liquid Quality Control (LQC) Test

(FR) Test de contrôle de la qualité des liquides. (DE) Flüssigkeitskontrolltest. (IT) Test di controllo qualità dei liquidi. (ES) Prueba de control de calidad de líquidos. (PT) Teste de Controlo de Qualidade Líquido. (CZ) Test kontrolní kvality kapalín.

(EN) COMPONENTS OVERVIEW

A. ANALYZER 1. Test Strip Eject Button. 2. Touch Screen Display. 3. Power Button. 4. Barcode Scanner. 5. Test Strip Port. 6. USB Port. **B. TEST STRIP** 7. Test Strip Insertion End. 8. Test Strip Barcode. 9. Sample Application Area. **C. TEST STRIP VIAL** 10. Test Strip Vial Barcode. **D. XPRECIA® SYSTEMS PT CONTROLS KIT** 11. PT Control 1. 12. PT Control 2. 13. CaCl₂ Diluent.

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(EN) 2. TURN ANALYZER ON. Hold finger on power button to turn analyzer on. 3. TAP "LQC" BUTTON. Tap LQC button to begin. Always wear gloves.

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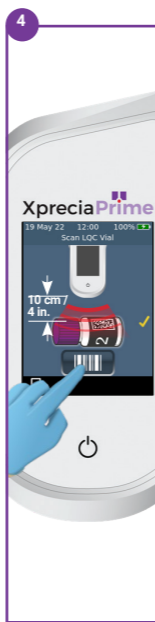
(EN) 4. SCAN PT CONTROL (LQC) VIAL. Tap on-screen barcode button to scan. Hold the barcode 10 cm (4 inches) from bottom of analyzer. 5. GET STRIP. Take strip from test strip vial, then close vial firmly so the lid clicks.

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(EN) 6. SCAN STRIP. When prompted, scan the test strip. If using vial for first time, the analyzer will also prompt you to scan the vial. 7. INSERT STRIP. Insert strip when prompted. Insert with arrows pointing towards analyzer.

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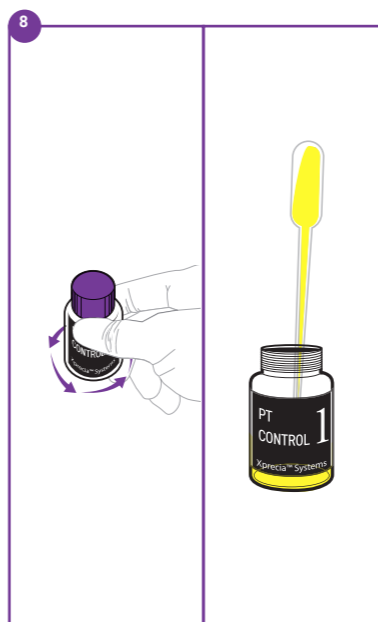
(EN) 8. GET PT CONTROL (LQC) SAMPLE (from Step 1). Swirl again briefly, then remove lid - be gentle to avoid bubbles. You must do the test within 25 mins of Step 1 (60 mins if refrigerated). Use pipette from Step 1.

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(EN) 9. APPLY PT CONTROL (LQC) SAMPLE. Only apply sample when prompted. Sample must fill whole of white semicircle.

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(EN) 10. RESULT. If result appears, LQC test passed. If error screen appears, see Troubleshooting in your User Guide. 11. EJECT STRIP. Eject strip as shown. All materials must go into a biohazard bin.

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(EN) 12. CLEAN ANALYZER. Turn over this sheet for cleaning instructions.

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(EN) WARNINGS & GUIDANCE:

- Always wear gloves when cleaning – the analyzer might be contaminated
- Clean after each and every test to ensure the analyzer is always safe to use
- Use only CaviWipes™ (to avoid damaging your analyzer)
- Make sure the CaviWipe™ is damp (not dripping): always squeeze out excess liquid into a biohazard bin
- Never let fluid gather around or enter the strip port – always keep the strip port pointing downwards during cleaning
- Before cleaning, disconnect the USB cable (if connected)

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(EN) **A. TURN ANALYZER OFF** Hold finger on power button and select power down to turn analyzer off. **B. QUICK WIPE OF ANALYZER** Use a fresh CaviWipe™ and then dispose of the wipe. **C. REMOVE STRIP PORT COVER.**

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(EN) **D. THOROUGHLY CLEAN ANALYZER** Use a fresh CaviWipe™ - squeeze or twist to remove excess liquid (damp not dripping). Do four vertical and four horizontal strokes on each surface (two up and down; two left and right)

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(EN) **E. CLEAN STRIP PORT COVER** Pay special attention to strip port and channels inside cover. **F. WAIT 2 MINUTES IMPORTANT:** The disinfecting properties of CaviWipes™ need this time to take effect

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(EN) **F. WAIT 2 MINUTES IMPORTANT:** The disinfecting properties of CaviWipes™ need this time to take effect

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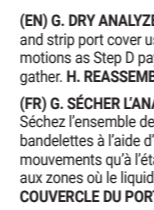
(EN) **G. DRY ANALYZER AND STRIP PORT.** Dry the whole analyzer and strip port cover using a lint free tissue/cloth. Use the same motions as Step D paying special attention to areas where fluid can gather. **H. REASSEMBLE STRIP PORT COVER**

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(EN) **J. CLEAN STRIP PORT COVER** Pay special attention to strip port and channels inside cover. **F. WAIT 2 MINUTES IMPORTANT:** The disinfecting properties of CaviWipes™ need this time to take effect

XpreciaPrime™

Liquid Quality Control Test

QUICK START GUIDE

©2022 Universal Biosensors. Xprecia Prime™ is a trademark of Universal Biosensors. All rights reserved. CaviWipes™ is a trademark of Metrex Research LLC. All other trademarks are the property of their respective owners.

Universal Biosensors
Designed by Universal Biosensors Pty Ltd
1 Corporate Avenue, Rowville, 3178, Victoria, Australia
79018 v2.1.1
Date of issuance September 2022

EC REP
MediMark Europe Sarl 11, rue Emile Zola, BP 2332 F- 38033 Grenoble Cedex 2, France
UK Responsible Person:
Obelis UK Ltd
Sandford Gate, East Point Business Park, Oxford OX4 6LB United Kingdom
UK CA